Upholstered Chairs/Leg Rests

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase this product from Aidapt. These products are manufactured from the finest material and components available. When installed and used correctly they are designed to give many years of reliable trouble-free service.

IMPORTANT!

N.B. This equipment must be installed by a competent person.

VG808/VG808B/VG808C/VG808G	Longfield Chair	140 kg (22 st.)
VG810/VG810B/VG810C/VG810G	Chelsfield Chair	140 kg (22 st.)
VG872	Winsham Bariatric Chair	254 kg (40 st.)
VG816/VG817	Welling Leg Rest	101 kg (16 st.)
VG818/VG819	Beaumont Leg Rest	101 kg (16 st.)

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

HEIGHT ADJUSTMENT

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2)

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

CLEANING

It is important that you only clean the upholstery using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair. Always ensure that you wipe dry the equipment after cleaning.

However the products in this group can be treated with formaldehyde if required, but it is important that this treatment be carried out by an approved contractor or an experienced person and under suitable conditions.

On no account must you attempt to clean with this method if you or your premises do not meet the legal requirements and guidelines governing the use of hazardous chemicals.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.


REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

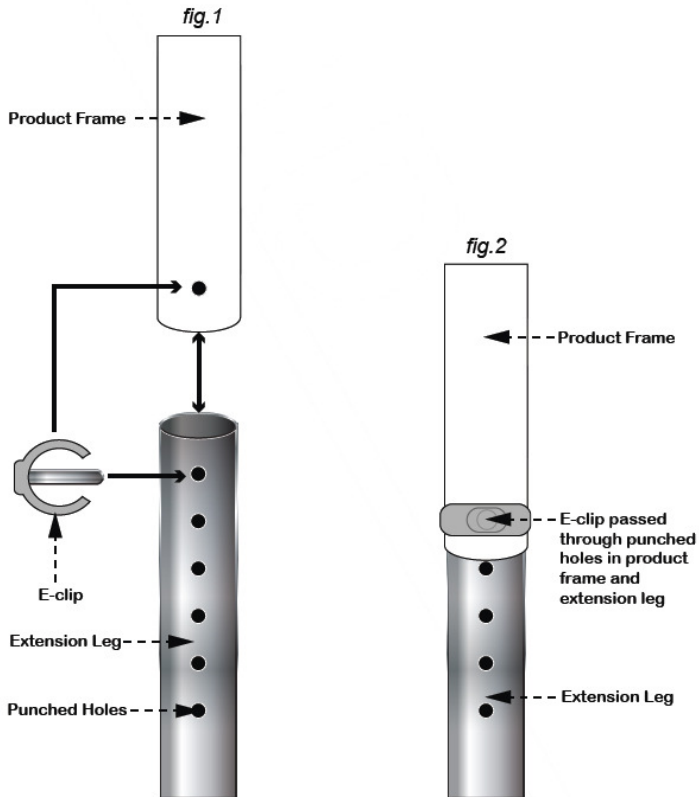IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Nederland (Altai Europe Ltd)