








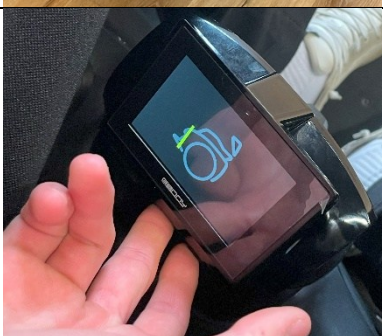

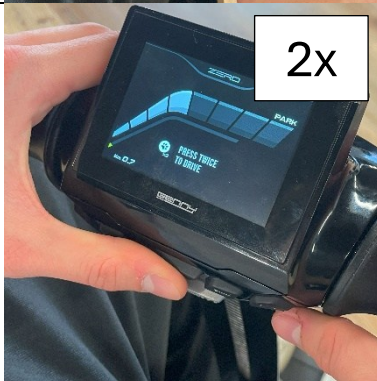


### HOW TO START GENNY ZERO MD AND DRIVE

1		<p>GENNY ZERO MD in “OFF” mode, with:</p> <ul style="list-style-type: none"> <li>- parking system disengaged</li> <li>- steering shaft at intermediate height</li> </ul> <p>If the batteries are not connected:</p> <ol style="list-style-type: none"> <li>Open the rear door and insert the batteries. The device will switch on automatically</li> </ol> <p>If the batteries are connected and the device is switched off:</p> <ol style="list-style-type: none"> <li>Press and hold the “STOP” button</li> </ol>
2		<p>The display switches on.</p> <p>At this point you can either</p> <ol style="list-style-type: none"> <li>Switch off the device by pressing and holding the ON/OFF button on the command cockpit display</li> </ol> <p>or</p> <ol style="list-style-type: none"> <li>Start the device: <ol style="list-style-type: none"> <li>position the device horizontally</li> </ol> </li> </ol> <p>The information on the screen changes to this.</p> 





3	 	<p>2. Press twice the “PAW” button to lower the legs</p> <p>The device is now in the “PARK” mode</p>
4	 	<p>3. Press and hold the “PAW” button to lower the steering shaft</p>


		The steering shaft moves downwards until it disappears
5		4. Transfer to GENNY
6		<p>5. Press and hold the “PAW” button to raise the steering shaft</p> <p>The steering shaft moves upwards until it stops</p>

7		<p>6. Press the central unlock button and open the handlebar grips</p> <p>The handlebar grips are now opened and horizontal</p>
8		<p>7. Press twice the “PAW” button to raise the legs</p> <p>The device is now in the driving mode “GUIDA”</p>







### HOW TO ACCESS THE QUICK COMMANDS MENU

1		1. Press repeatedly the “STAR” button to access the pages available on the quick commands’ menu
2		1 <sup>st</sup> page: <ul style="list-style-type: none"> <li>- Activation of the yellow direction arrows</li> <li>- Activation of the 4 yellow emergency arrows</li> </ul>
3		2 <sup>nd</sup> page: <ul style="list-style-type: none"> <li>- ON/OFF switch for the headlights</li> </ul>
4		3 <sup>rd</sup> page: <ul style="list-style-type: none"> <li>- Activation of the “TURTLE” mode (speed limitation below 5km/h)</li> </ul>

5		<p>4<sup>th</sup> page:</p> <ul style="list-style-type: none"> <li>- Balance tilt angle setting (use to adjust the static balance position)</li> </ul>
6		<p>Notes:</p> <ul style="list-style-type: none"> <li>- At a speed above 3km/h, the menu is not accessible</li> <li>- While driving above 3km/h, direction arrows can be activated by swiping a finger horizontally to the left or to the right across the display</li> </ul>

### HOW TO PARK GENNY ZERO MD

1	 	<ol style="list-style-type: none"> <li>1. Slow down and stop</li> <li>2. Find a static balance position</li> <li>3. Press twice the “PAW” button to lower the legs</li> </ol> <p>The device is now in the “PARK” mode</p>
2		<p>Note:</p> <ul style="list-style-type: none"> <li>– You can now lower the steering shaft and transfer from GENNY ZERO MD</li> </ul>
3	 	<p>When no user is on board, the ON/OFF button will become available on the cockpit display</p> <p>To switch GENNY ZERO MD off:</p> <ol style="list-style-type: none"> <li>1. Press the ON/OFF button on the display</li> <li>2. Wait a few seconds</li> </ol>
4		<p>Note:</p> <ul style="list-style-type: none"> <li>- If not used, the device will shut down automatically after a few minutes if no user is on board</li> </ul>


### ENABLING THE “SERVOGUIDA” MODE

1



1. Put GENNY ZERO MD in the “PARK” mode with the paws lowered
- The steering shaft can be either in a high or low position

2

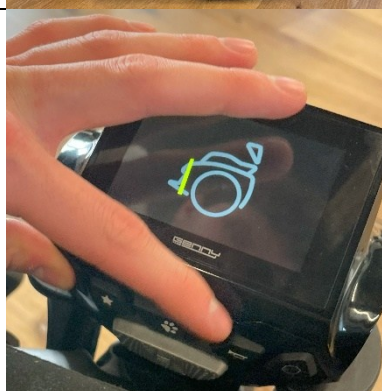



2. Do press simultaneously:
- The “STOP” button (and hold it)
- The “PAW” button (twice)



The device is now in “OFF” mode, with the paws raised and the steering shaft at an intermediate height.

This setting allows the user to move the device freely with the engines switched off.



3		3. Press the seating cushion
4		<p>The device is now in the “SERVOGUIDA” mode:</p> <ul style="list-style-type: none"> <li>- Engines are on and GENNY ZERO MD keeps itself in balance with no users on board</li> <li>- The user can move the device just by pushing softly on the steering</li> </ul>
5		<p>Note:</p> <ul style="list-style-type: none"> <li>- To go back to the “PARK” mode:</li> </ul> <ol style="list-style-type: none"> <li>1. Push the “STOP” button</li> <li>2. Hold the device horizontally</li> <li>3. Press twice the “PAW” button to lower the paws</li> </ol>