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© 2016 Sunrise Medical

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Sunrise Medical.

The information provided is based on general data concerning the construction known at the time of the publication of this manual. Sunrise Medical executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Sunrise Medical can therefore not be held liable for any damage resulting from specifications of the product deviating from the standard configuration. Illustrations contained in the manual may deviate from the configuration of your product.

The information made available has been prepared with all possible diligence, but Sunrise Medical cannot be held liable for any errors contained in the information or the consequences thereof.

Sunrise Medical accepts no liability for loss resulting from work executed by third parties.

Names, trade names, trademarks etc. used by Sunrise Medical may not, as per the legislation concerning the protection of trade names, be considered as being available.

1	General safety guidelines and instructions.....	16
2	Operating instructions	17
2.1	Controller	17
2.2	Charging the batteries.....	18
2.3	Parking.....	19
2.4	Driving with PowerSupport assistance.....	19
2.5	Braking.....	19
2.6	Switching off.....	19
2.7	Curves (tip)	19
2.8	Slope.....	20
2.9	Obstacles.....	20
3	Maintenance en troubleshooting.....	21
4	Technical specifications	22
5	Warranty and liability.....	23

Important: Please study the owner’s manual carefully before using the PowerSupport. This manual is a supplement to the Ibis owner’s manual.

1 General safety guidelines and instructions

The manufacturer accepts no liability whatsoever for damage or injury caused by failure to (strictly) adhere to the safety guidelines and instructions or else due to carelessness during the use and cleaning of the PowerSupport module. Depending on the specific work circumstances, additional safety instructions may be required. Please contact your dealer immediately if you observe a potential danger during the use of the product.

- **To avoid accidents and other problems, it is very important to pay attention to the safety instructions as described in the Ibis owner's manual of Sunrise Medical.**
- **The PowerSupport should only be used for the transport of persons to a maximum weight of 125 kg. Tests were conducted with this user weight.**
- **When charging the batteries it is strongly advised to use a charger which complies with the specifications on page 10 of this manual.**
- **When the wheelchair is not being used, it is advised to connect the battery charger to the PowerSupport system. It is best to charge the batteries every night.**
- **Never use the PowerSupport on a slope with an angle of more than 6°.**
- **With a user weight of more than 100 kg the wheelchair will hardly brake, if at all, when driving down a slope.**
- **Always use the parking brake when parking the PowerSupport.**
- **Do not connect any other equipment to the PowerSupport.**
- **Never open the battery cover.**
- **Never clean the PowerSupport using a hose or high-pressure spraying pistol. The PowerSupport should be cleaned with a damp cloth.**
- **Be careful when (un)locking the parking brake. The moving parts of the braking system may cause jamming.**
- **The PowerSupport is suitable for indoor and outdoor use (EN 12184 (1999) class B). Driving through deep puddles is not considered as normal use and may cause damage to the motors.**

Parts list, electrical connection diagrams and detailed repair instructions are available on request.

2 Operating instructions

2.1 Controller

On/Off indicator

Green indicator light which is on when the PowerSupport is activated. In case of a general malfunction the indicator will lighten up.

Operating handle

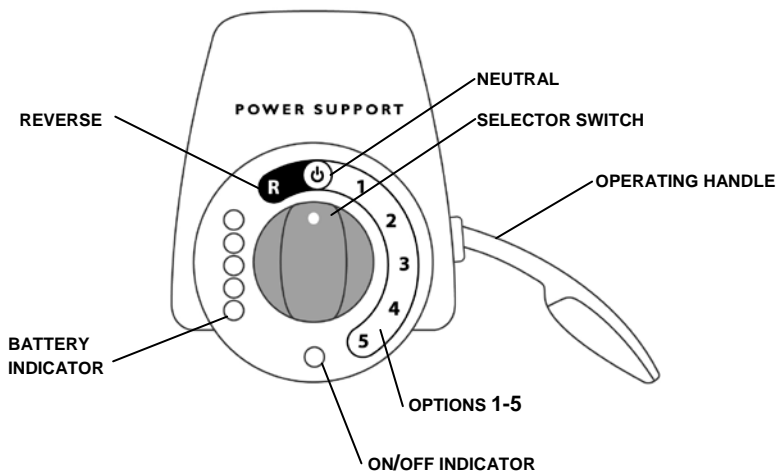
Handle (on/off) which activates the power assistance function.

Selector switch

Button on the controller to set the power supply. With this button, the walking speed can be set. The selector switch has seven fixed positions: reverse (R), neutral (off mode), and five drive forward positions (1 - 5).

Battery indicator


Lights showing the battery condition. The battery indicator comprises 5 lights (red: 20%, orange: 40%, green: 60, 80 and 100% charge). When the PowerSupport is activated or is being charged, the battery indicator is visible. When the battery charge is less than 40%, you should charge the batteries.

2.2 Charging the batteries

The PowerSupport may be charged at any time. The charging process may also be interrupted at any time. As soon as the battery charger is connected to the PowerSupport, the battery indicator will show the battery condition. During charging the PowerSupport is protected and it is not possible to drive with PowerSupport assistance.

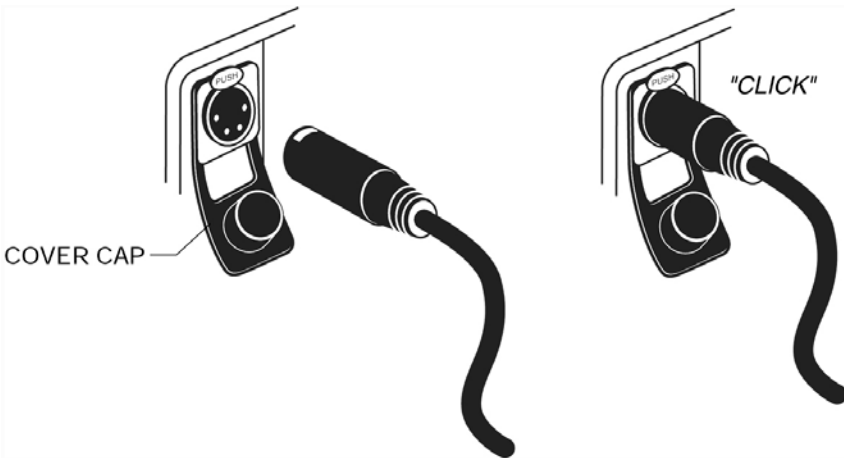During charging one of the battery indicator lights will flash. The battery charge level can be recognized by the number of lights that are constantly lit. When the battery is fully charged, all indicator lights will stay on.

1. Put the charge cable of the battery charger in the wall outlet of the electricity mains.
2. Remove the cover cap from the battery pack.
3. Put the four-pin charge plug of the battery charger in the battery pack until you hear a clear 'click'.
4. Check if a battery indicator light on the controller is flashing when you start charging the batteries. This indicates that the batteries are actually being charged.
5. After charging, check if the battery indicator lights on the controller are permanently lit. This indicates that the batteries have been charged properly.
6. Push back the release catch and remove the charge plug of the battery charger from the battery pack. Attention: the charge plug cannot be removed from the battery pack if the release catch is not pushed in. This may damage the electronics.
7. Put the cover cap of the battery pack back in place.
8. The PowerSupport is now ready for use.

**It is advised to mount a smoke detector in the charging area.
Disconnect batteries in case of longer storage without usage.**

2.3 Parking

When the PowerSupport is not in use, put the selector switch in neutral (off) and engage the parking brake of the Ibis wheelchair. When the PowerSupport is used, release the parking brake.

Be careful when (un)locking the parking brake. The moving parts of the braking system may cause jamming.

2.4 Driving with PowerSupport assistance

Put the selector switch in the desired position (R, for reverse position or position 1-5 for forward). Make sure you choose a power level that suits the desired walking speed. Bear in mind the surface. A smooth surface requires less assistance than a slope or rough surface, such as carpet.

Put the selector switch in neutral to move the Ibis without assistance (position 0).

2.5 Braking

When driving with assistance, the PowerSupport will actively slow down the wheelchair when the operating handle is released.

When driving with assistance (selector switch in position 1-5, operating handle not engaged), the motor will automatically brake when the speed exceeds 3 km/h.

When driving with assistance (selector switch in position 1-5, operating handle not engaged), the motor will automatically brake when the wheelchair drives backward, ensuring increased safety on a slope.

2.6 Switching off

The PowerSupport does not have a separate on/off switch. The PowerSupport will switch off automatically, if it has not been used for a while (5 minutes in position R or 1-5). In position 0, neutral, the PowerSupport will switch off immediately. As soon as the operating handle is pushed forward, or the selector switch is turned, the PowerSupport is activated.

2.7 Curves (tip)

Just keep the operating handle engaged when taking a curve. If you release the operating handle, the wheelchair brakes actively, making it more difficult to drive in a curve.

2.8 Slope

Only drive with PowerSupport assistance on slopes you are familiar with. Slopes with an angle of more than 6° are not suitable for the PowerSupport.

The top 3 positions are suitable for driving on a slope with assistance.

Driving up a slope

- Stop before the slope.
- Before driving up the slope, put the selector switch in a position providing sufficient assistance (preferably position 4/5).
- If necessary change to a higher position if you feel the selected position is not providing sufficient assistance.
- Avoid neutral (off mode) and reverse (R). In these positions the wheelchair does not brake so the attendant receives the full weight.

Driving down a slope

- Stop before the slope.
- Before driving down the slope, put the selector switch in the forward position (1-5). Do not use the operating handle, but push the wheelchair down the slope by yourself. The motors will brake slightly if the speed exceeds 3 km/h.
- Avoid neutral (off mode) and reverse (R). In these positions the wheelchair does not brake so the attendant receives the full weight.

Attention: with a user weight of more than 100 kg the wheelchair will hardly brakes, if at all.

2.9 Obstacles

The PowerSupport is suitable for kerbs up to 4 cm. Drive carefully towards the kerb with the front wheels. Put the PowerSupport in position 4 or 5. Then take a short run up (15 cm) to drive up the kerb. Make sure the wheelchair does not carry on accelerating after mounting the kerb.

3 Maintenance en troubleshooting

Frequency	Maintenance
Daily	Check: - does the parking brake work properly? - flat tyre?
Weekly	Check tyre pressure and moving parts of the parking brake.
Annually	Full service by service engineer, adjust parking brake.

Contact your dealer for repair work. A service manual is available for the dealer.

- Never clean the PowerSupport with a hose or high-pressure spraying pistol. The PowerSupport should be cleaned with a damp cloth, if necessary with some household soap dissolved in water.
- **Attention! In case of a malfunction, the PowerSupport system will not brake. This may cause a dangerous situation on a slope, as the attendant receives the full weight of the wheelchair and user. Therefore, immediately engage the parking brake when identifying a malfunction.**
- In case of a malfunction, the on/off indicator will start flashing. This is a general malfunction indication.
- **Action:** check all motor connections. If necessary secure the plug connection of the motor. In any other case, contact the service engineer.
- If the lights/indicators on the control unit do not blink:
Action 1: check if the control unit cable is loose at the plug connection to the battery pack.
Action 2: connect the battery charger, lift the wheelchair and carefully turn one of the wheels. The PowerSupport should start charging. In any other case, contact the service engineer.
- If the battery indicator lights are flashing or lit constantly, the PowerSupport is probably still charging. Remove the plug from the battery charger. The PowerSupport is now ready for use.
- In case of a flat tyre, the service engineer should be contacted. If you continue to drive, you may damage the wheels.

4 Technical specifications

Type: PowerSupport Ibis

Tested by: TNO, in conformity with EN 12184 (1999) class B, for indoor and outdoor use.

Weight test dummy: 125 kg.

IBIS with PowerSupport

Weight: 55.6 kg

Turning radius: 770 mm

Maximum width: min 687, max 720mm

Static stability for IBIS with PowerSupport

Upward: 11°

Downward: 10°

Sideways: 15°

Dynamic stability upward: 9°

Recommended maximum angle on a slope upward and downward: 6°.

Range: 18-20 km

Maximum speed: 5 km/h

The PowerSupport has its own battery system. The battery cover is not to be opened.

Battery type: NiMH, 24 V

Battery capacity: 9 Ah

Charging time: 2 hours for a full charge.

Specifications of a suitable battery charger:

Input 220V AC

Output 24 V DC, 3.4 A +/- 5%

XLR connector 4-pin

A suitable battery charger can be provided by Sunrise Medical

Tyre size: 12.5x2 1/4"

Tyre pressure: 2.7 Bar

5 Warranty and liability

The PowerSupport is TNO approved in conformity with the requirements of the European wheelchair standard EN 12184 (1999) class B.

The warranty and liability stipulations as described in the Ibis owner's manual are applicable to the Ibis with PowerSupport.

Contrary to the above mentioned rule, the following applies to the PowerSupport module:

The batteries carry a two year warranty, under the following conditions:

- Always use the Ibis with PowerSupport in conformity with the instructions in this owner's manual.
- Never open the battery cover.
- Do not use the Ibis PowerSupport at a temperature of less than -10° Celsius.
- Do not use the Ibis PowerSupport at a temperature of more than 40° Celsius.
- Do not charge the batteries at a surrounding temperature of less than 0° Celsius.
- Do not charge the batteries in direct sunlight or at a surrounding temperature of more than 40° Celsius. Note that in direct sunlight the temperature in the battery cover can easily rise beyond this temperature.

Description	Warranty period	Examples include, but are not limited to the parts mentioned below
Drive system*	1 year	Motor, motor mechanics
Electronics*	1 year	Controller, controlling mechanism, wiring harness, electronic components
After sales service parts	New: 1 year after invoice Repaired: 90 days after invoice	Brakes
Consumable parts	40 days after invoice	Spoke guards, grip friction pushrim cover
Options/Accessories	1 year	Delivered with the initial product

* also in case of after sales service part delivery

Indes B.V.
P.O. Box 265
7500 AG Enschede
The Netherlands
www.indes.eu

The PowerSupport is a product of Indes Production Management B.V. and is in conformity with the provisions of the medical devices directive concerning class I medical devices.

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AU
Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood
NSW 2148
Australia
T: +61 (2) 9678 6600
E: enquiries@sunrisemedical.com.au
www.SunriseMedical.com.au

BE
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +32 (0)78 150 159
E: customerservice@sunrisemedical.be
www.SunriseMedical.be

CA
Distributed in Canada by:
Sunrise Medical Canada Inc.
237 Romina Drive, Unit 3
Concord, ON
L4K 4V3, Canada
T: +1 800 263 3390
E: mktcanada@sunmed.com
www.SunriseMedical.ca

CH
Sunrise Medical AG
Lückhalde 14
3074 Muri/Bern
Schweiz/Suisse/Svizzera
T: +41 (0)31 958 3838
E: info@sunrisemedical.ch
www.SunriseMedical.ch

DE / AT
Sunrise Medical GmbH & Co. KG
Kahlbachring 2-4
68254 Malsch/Heidelberg
Deutschland
T: +49 (0) 7253/980-0
E: kundenservice@sunrisemedical.de
www.SunriseMedical.de

DK
Sunrise Medical ApS
Park Allé 289 G
DK-2605 Brøndby
T: +45 70 22 43 49
Email: info@sunrisemedical.dk
www.sunrisemobility.dk

ES / PT
Sunrise Medical S.L.
Poligono Bakiola, 41
48498 Arrankudiaga – Vizcaya
España
T: +34 (0) 9021 42434
E: marketing@sunrisemedical.es
www.SunriseMedical.es

FR
Sunrise Medical S.A.S
ZAC de la Vrillonnerie
17 Rue Mickaël Faraday
37170 Chambray-Lès-Tours
T: + 33 (0) 2 47 55 44 00
E: info@sunrisemedical.fr
www.SunriseMedical.fr

IT
Sunrise Medical S.r.l.
Via Riva, 20 – Montale
29122 Piacenza
Italia
T: +39 0523 573111
E: info@sunrisemedical.it
www.SunriseMedical.it

NL
Sunrise Medical HCM B.V.
Vossenbeemd 104
5705 CL Helmond
The Netherlands
T: +31 (0)492 593 888
E: customerservice@sunrisemedical.nl
www.SunriseMedical.nl
www.SunriseMedical.eu (International)

NO
Sunrise Medical AS
Dynamitveien 14B
1400 SKI
Norway
T: +47 66963800
E: post@sunrisemedical.no
www.SunriseMedical.no

PL
Sunrise Medical Poland Sp. z o.o.
ul. Elektronowa 6,
94-103 Łódź
Polska
T: + 48 42 275 83 38
E: pl@sunrisemedical.de
www.Sunrise-Medical.pl

SE
Sunrise Medical AB
Box 9232
400 95 Göteborg
Sweden
T: +46 (0)31 748 37 00
E: post@sunrisemedical.se
www.SunriseMedical.se

UK
Sunrise Medical
Thorne Road
Brierley Hill
West Midlands
DY5 2LD
United Kingdom
T: +44 (0) 845 605 66 88
E: enquiries@sunmed.co.uk
www.SunriseMedical.co.uk

Dealer / distributor:

Serial number:

OM_PowerSupport_EU_NL_EN_FR_RevH_2016_05_15