# Invacare® Softform®

en	Mattresses User Manual .....	3
de	Matratzen Gebrauchsanweisung .....	17
fr	Matelas Manuel d'utilisation .....	33
es	Colchones Manual del usuario .....	49
it	Materassi Manuale d'uso .....	65
nl	Matrassen Gebruiksaanwijzing .....	81
pt	Colchões Manual de utilização .....	97
sv	Madrasser Bruksanvisning .....	113
no	Madrasser Bruksanvisning .....	127
da	Madrasser Brugsanvisning .....	141

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, read this manual and save for future reference.

**Yes, you can.®**

©2017 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

---

# Contents

---

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1 General</b>	<b>4</b>
1.1 General information	4
1.2 Symbols in this user manual	4
1.3 Compliance	4
1.4 Warranty	5
1.5 Limitation of liability	5
1.6 Intended use	5
1.7 Service life	5
<b>2 Safety</b>	<b>6</b>
2.1 Safety information	6
2.2 Symbols on the product	7
<b>3 Components</b>	<b>8</b>
3.1 Product description	8
3.2 Components	8
<b>4 Usage</b>	<b>9</b>
4.1 Safety information	9
<b>5 Transport</b>	<b>10</b>
5.1 Safety information	10
<b>6 Maintenance</b>	<b>11</b>
6.1 Inspection	11
6.2 Cleaning and care	11
<b>7 After Use</b>	<b>13</b>
7.1 Storage	13
7.2 Re-Use	13
7.3 Disposal	13
<b>8 Technical Data</b>	<b>14</b>
8.1 General Data	14

8.2 Materials	15
8.3 Environmental Parameters	15

# 1 General

## 1.1 General information

Essential nursing care is pivotal in pressure ulcer prevention. These mattresses will positively contribute to the outcome of a pressure ulcer prevention care plan.

Education, clinical judgement and action based planning based on vulnerability are fundamental factors in the prevention of pressure ulcers.

A range of assessment scales can be used as a formal method of assessing risk from pressure ulcer development, and should be used in conjunction with an informal assessment (informed nursing judgement). Informal assessment is considered to be of greater importance and clinical value.

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

For further information contact Invacare in your country (addresses see back page of this user manual).

To access valuable information and useful links for Pressure Area Care training and education, refer to [www.thinkpressurecare.co.uk](http://www.thinkpressurecare.co.uk).

## 1.2 Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

### WARNING

Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

### CAUTION

Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

### IMPORTANT

Indicates a hazardous situation which if not avoided could result in damage to the product.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

## 1.3 Compliance

Quality is fundamental to the company's operation, working within the discipline of ISO 13485.

All Invacare Softform Mattress products feature the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally,

is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (addresses see back page of this manual).

## 1.4 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Guarantee claims can only be made through the provider from whom the appliance was obtained.

## 1.5 Limitation of liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorized modifications and/or use of unsuitable spare parts

## 1.6 Intended use

This pressure redistribution mattress is intended to be used in conjunction with an appropriately sized bed frame, as part of an overall pressure ulcer prevention program of care.

Any other use is prohibited.

This product has been designed to deliver effective pressure redistribution to users, when the product is in normal use

which is defined by Invacare as when the support surface is covered with a cotton, cotton combination or linen bed sheet, and any one of these would be the only item deployed between the support surface and the user.

## 1.7 Service life

We estimate a life expectancy of five years for these products, provided they are used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage.

The fact that we estimate a life expectancy for these products does not constitute an additional warranty.

## 2 Safety

### 2.1 Safety information

**WARNING!**

Do not use this product or any available optional equipment without first completely reading and understanding the user manual supplied. Invacare product manuals are available on your local Invacare website or at your local dealer. If you are unable to understand the warnings, cautions or instructions please contact a health care professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

**WARNING!****Risk of developing pressure ulcers**

Bed sheets must be loosely fitted, with creases smoothed out. Care must always be taken to ensure that the support surface in contact with the user is kept free from crumbs and other food debris, and that drip cables, stents, and other foreign objects do not become entrapped between the user and the pressure reducing surface of the mattress, as this may result in the development of pressure ulcers.

**WARNING!**

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

The introduction of certain third party products between the mattress surface and the user may reduce or impede the clinical effectiveness of this product.

'Third party products' may include, but are not limited to items including under blankets, plastic sheets and sheepskins, etc. Heated over blankets must only be used in consultation with a suitably qualified health care professional, as an increase in temperature can increase the risk of developing pressure ulcers.

**WARNING!****Risk of fire or explosion!**

A cigarette can burn a hole in the bed surface and cause damage to the mattress. Also, patient clothing, bed sheets, etc, may be combustible and cause a fire. Failure to observe this warning can result in a severe fire, property damage and cause physical injury or death.

- Do not use in oxygen rich environments.
- Do not smoke.


### IMPORTANT!

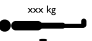The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.
- In case of damage, do not use.
- Contact Invacare for further guidance/information.

## 2.2 Symbols on the product

	Do not pierce or cut		Line dry
	Do not dry clean		CE conform
	User weight limit*		Recommended 80 °C
	Do not put near flame		Do not iron
	Do not bleach		Tumble dry low heat
	Date of manufacture		

\* Maximum user weight as per section 8 *Technical Data*, page 14.

### 3 Components


#### 3.1 Product description

The Invacare® Softform® mattress offers exceptional pressure reducing qualities, specific to the individuals needs. By promoting even weight distribution, they reduce tissue pressure at potentially vulnerable areas.

The water-resistant cover provides a vapour-permeable, multi stretch surface, to promote patient comfort and to maximize the effectiveness of the foam core.

#### 3.2 Components

The following components are included within the scope of delivery \*:

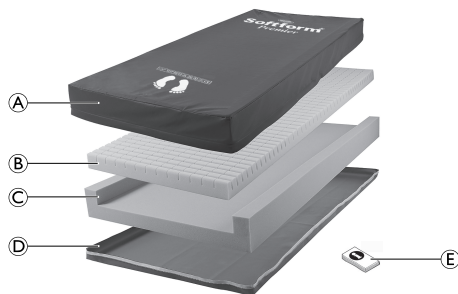(A)	Multi-stretch waterproof polyurethane cover
(B)	Castellated foam insert
(C)	Single piece foam "U" core

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\* This example is based on the Softform® Premier, other mattress configurations may vary.

## 4 Usage

### 4.1 Safety information

1. Remove all packaging before use.
2. Place the mattress directly on the frame of the bed.

The mattress is designed for beds with adjustable lying surface.

#### **WARNING!**

It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This must be based on the clinical judgement of a qualified health care professional. This relieves pressure which helps prevent both tissue compression and potential ulcer formation.

- Always consult a qualified health care professional before using the product.
- Monitor the patient frequently.

#### **CAUTION!**

- Make sure that the printed side of the mattress cover always faces upwards.
- Ensure that the distance between the surface of the mattress and the top of the side rail is at least 220 mm. If this is not achievable a risk assessment must be carried out.

#### **IMPORTANT!**

- If holes are present in the mattress cover, there is a danger that liquids may ingress and contamination may occur.
- Medical equipment including infusion pumps and monitors should be attached to appropriate bed accessories.
  - For home use common causes of damage include cigarette burns and the claws of pets that puncture sheets, allowing fluid ingress and staining.

#### **IMPORTANT!**

##### **Risk of damage to the mattress cover**

- To prevent accidental damage, do not place hypodermic needles, venflons, scalpels or other similarly sharp objects onto the mattress.
- Ensure that all venflons are taped down correctly with no sharp edges exposed.
- When using patient transfer aids, care should be taken not to damage the mattress. All transfer aids should be checked for any sharp edges or burrs before use as these can damage the mattress.
- Make sure that the mattresses are not jammed or damaged by sharp edges when used on beds with an adjustable frame.
- When using the mattress on a profiling bed ensure that the knee break is used before the backrest.

---

## 5 Transport

---

### 5.1 Safety information

#### **IMPORTANT!**

- Take care when handling mattresses to ensure no damage to the cover. It is recommended that two people lift/carry mattresses.
- Avoid contact with jewellery, nails, abrasive surfaces etc.
- Do not drag mattresses.
- Avoid contact with wall, door frames, door catches or locks etc.
- Do not transport in roll cages unless completely protected from the sharp edges of the cage.

1. Refer to the storage and shipping conditions in section *8.3 Environmental Parameters, page 15.*

## 6 Maintenance

### 6.1 Inspection

Check mattresses (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

#### Check mattress

1. Unzip the cover completely.
2. Check for any staining on the white underside of the cover.
3. Check for any staining on the interior foam.
4. Replace any stained items and dispose of as per local authority procedure.

### 6.2 Cleaning and care

- ! **IMPORTANT!**  
All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
  - For further information on decontamination in health care environments, please refer to ‘The National Institute for Clinical Excellence’ guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local country infection control policy.

#### Cleaning covers

(Removal of contaminants such as dust and organic matter)

PALIN-010-C

1. Remove all covers for laundering.
2. Launder the covers with the recommended temperature at 80 °C using a diluted detergent solution (Instructions on label).

- ! **IMPORTANT!**  
Washing at higher temperatures will cause shrinkage.

#### Drying covers

1. Hang mattress covers from a line or bar and drip dry in a clean indoor environment.  
or  
Tumble dry on a low heat setting.

- ! **IMPORTANT!**
  - Tumble dry setting must not exceed 40 °C.
  - Do not tumble dry for longer than 10 minutes.
  - Dry thoroughly before re-fitting to the foam.

#### Disinfecting covers

(Reducing the number of microorganisms)

Please contact your hygiene specialist in the event of contamination.

- ! **IMPORTANT!**
  - Ensure that any residual detergent has been removed with clean water prior to disinfection.

#### Light soilage

1. Wipe down the cover with a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the cover with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

### Heavy soilage

Where the mattress is badly soiled, we recommend cleaning with a dilute cleaning solution at 80 °C in the washing machine.

 Large spillages of blood should be absorbed and removed with paper towels first, followed by as above.

1. Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a 1% Chlorine Solution (10,000 ppm).
2. Rinse with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

**!** **IMPORTANT!**  
Polyurethane coated fabrics can absorb liquids for short periods causing a temporary change to the polyurethane characteristics. The mattress cover swells temporarily and is more vulnerable to physical damage for a period after it is completely surface dried, by which time it will revert to its previous state.

**!** **IMPORTANT!**  
1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly.  
– Do not use granules.

 **WARNING!**  
– Remove contaminated foams from use.

 **CAUTION!**  
– Keep clear of open heat sources.

**!** **IMPORTANT!**  
– Do not use phenols, alcohols, bleaches, or other abrasive materials.

### Replacing covers

1. Unzip the cover and remove it carefully from the foam core.
2. Place new cover onto the foam core.
3. Then close the zipper.

**!** **IMPORTANT!**  
– Ensure that the corners of the foam core are positioned correctly into the corners of the cover.  
– Ensure that the castilled foam is facing uppermost when packed into its cover.

---

## 7 After Use

---

### 7.1 Storage

To ease storage and handling the mattresses are sometimes compact rolled depending on customer requirements. This in no way affects the performance and longevity of the product. When the mattress is unrolled for the first time, allow it to recover for a minimum of 1 hour before use to optimize performance.

#### **IMPORTANT!**

- Store mattresses in a dry environment.
- Store mattresses within a protective cover.
- Store items on clean, dry, off-flooring free from sharp edges to avoid any possible damage.
- Never store other items on top of a mattress.
- Do not store mattresses next to radiators or other heating devices.
- Protect mattresses from direct sunlight.

1. Refer to the storage and shipping conditions in section *8.3 Environmental Parameters, page 15*.

### 7.2 Re-Use

A cleaning record must be kept as part of cleaning the system.

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

1. Before reuse, clean the product thoroughly,  
→ *6.2 Cleaning and care, page 11*.

### 7.3 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

1. Ensure that the mattress is cleaned prior to disposal to avoid any risk of contamination.

## 8 Technical Data

### 8.1 General Data

Product	Firetesting	Grade ref & Colour	Nominal density range [kg/m <sup>3</sup> ]	Nominal hardness range [N]	Maximum user weight [kg]	Weight of product [kg] <sup>1)</sup>
PREMIER	EN 597-1, 2 BS 7177: Crib 5	RX 39/120 Pink RX 39/200 Blue	38 – 40 38 – 40	105 – 135 180 – 200	247.6	14
MAXIGLIDE	EN 597-1, 2 BS 7177: Crib 5	RX 39/120 Pink RX 39/200 Baltic Blue	38 – 40 38 – 40	105 – 135 180 – 200	247.6	13.5
PREMIER VISCO	EN 597-1, 2 BS 5852: Crib 5	RX 39/120 Pink RX 39/200 Blue VC 55085 White	38 – 40 38 – 40 55	105 – 135 180 – 200 85	247.6	14
EXCEL	EN 597-1, 2 BS 7177: Crib 5	RX 39/120 Pink RX 39/200 Blue	38 – 40 38 – 40	105 – 135 180 – 200	140	10
PREMIER ORIGINAL	EN 597-1, 2 BS 7177: Crib 5	RX 39/120 Orange RX 39/200 Blue	38 – 40 38 – 40	105 – 135 180 – 200	247.6	13
BARIATRIC MATTRESS	EN 597-1, 2 BS 7177: Crib 5	RX 39/120 Pink RX 39/200 Blue	38 – 40 38 – 40	105 – 135 180 – 200	340	15
A&E TROLLEY MATTRESS	EN 597-1, 2 BS 5852: Crib 5	RX 36/125 Beige RX 39/200 Blue	35 – 37 38 – 40	110 – 140 180 – 200	108 (TM20/TM40) 140 (TM60)	5.5

<sup>1)</sup> Based on the weight of a standard size mattress. This can change if different sizes are ordered.

## 8.2 Materials

<b>Invacare® Softform®</b>	
Foam	Polyurethane Combustion Modified High Resilience Foam
Cover	Polyurethane transfer coating on weft knitted fabric
Cover (A&E Trolley mattress)	Polyurethane transfer coating on weft knitted fabric with enhanced electrical conductivity

All mattress components are not made from natural rubber latex.

## 8.3 Environmental Parameters

<b>Operating conditions</b>	
Ambient temperature	10 - 35 °C
Relative humidity	30% - 75% Non-Condensing
Atmospheric pressure	70 - 106 kPa
<b>Storage and shipping conditions</b>	
Ambient temperature	-40 - 70 °C
Relative humidity	10% - 100%
Atmospheric pressure	50 - 106 kPa

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Invacare AB