



BELLAVITA BATHLIFT DISASSEMBLY GUIDE

In the event of the Bellavita being stuck in the upright position, after gaining approval from Drive Medical a dealer, issuer or user may disassemble the bathlift to allow easier transportation and/or storage back to Drive Medical. Please follow the steps below:




Step 1. Lay the bathlift on its side.

Remove the new backrest from the carton.



Step 2. Using a T-20 end bit and a screwdriver, remove all the screws on the backrest.

(T-20 end bits are available from free from Drive Medical on request)

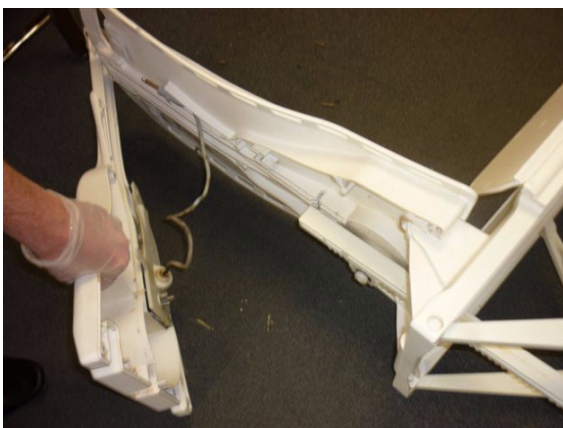

Step 3. Pull the rear of the backrest away from the bathlift.



Step 4. Remove the lifting ram and the faulty backrest from the bathlift.
Place the faulty backrest in the carton and return to Drive Medical.