


BACKREST ADJUSTMENT




Loosen the knobs both left and right from the backrest. Adjust the backrest to a desired position and tighten the knobs again.

ARMRESTS

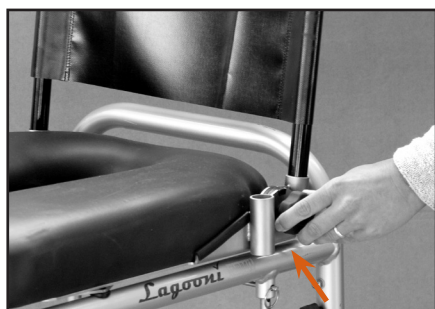

Slightly lift the armrests to swivel them around. When placing them back, make sure the armrests will be tightened by pushing them down well enough.

FOOT- AND LEGRESTS

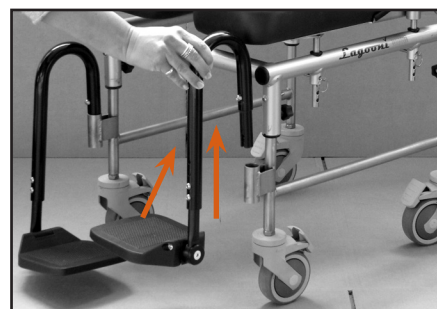


Lift the legrests and swivel them around. It is possible to fold up the footrests. Use allen key nr. 4 to adjust the height of the footrests.

SEAT POSITION




Remove the 4 quick-release pins from under the seat.

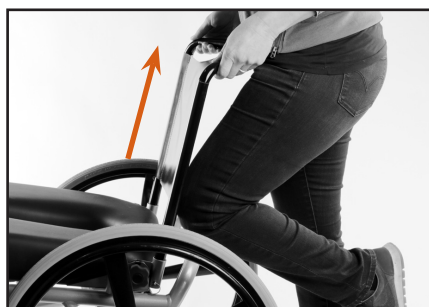


Lift the backrest and seat to a desired position. Replace the quick-release pins.

ANGULAR ADJUSTMENT

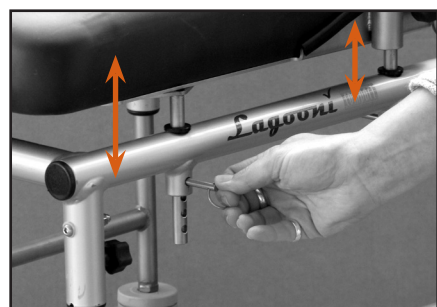

Push the seat max. one pin up on the front of the seat comparing to the back to create an angular adjustment. Replace the pins at front and push the back down.

DISMANTLING

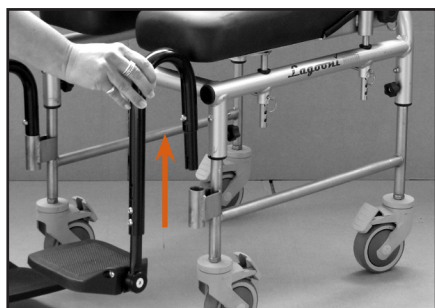


Remove the legrests by lifting them up.



Remove the armrests by lifting them up.



Remove the quick release pins and put the seat into the lowest position. Replace the quick release pins.

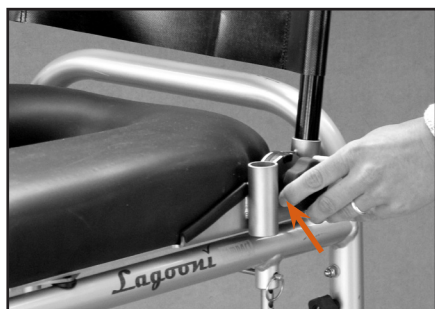

Loosen the knobs of the backrest and remove the quick-release pins to turn the backrest.



Turn the backrest 180 degrees and make sure the bending faces front. Replace the backrest and quick-release pins.




Fold down the backrest and tighten the knobs.



When applicable, remove the rear wheels by pressing the quick-release axles.



Remove the 4 small knobs from the legs of the wheelchair.

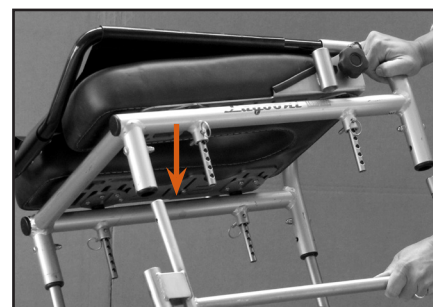


Remove the legs from the frame by pulling them out.


PACK IN THE SUITCASE




Open the suitcase and place something inside to protect the wheelchair. Unzip the inside at least halfway.

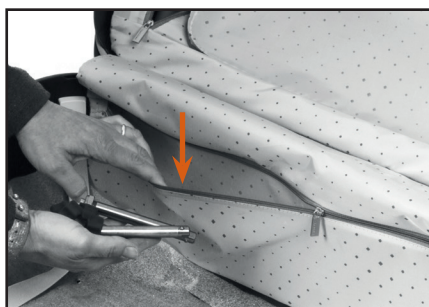


Place the knobs, axles and other small objects in the side of the suitcase.

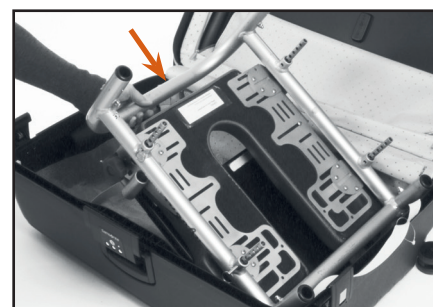


Place the frame with the seat facing down in the suitcase. If the width is 46 cm or more, turn the seat 90 degrees and place these separate from the frame.

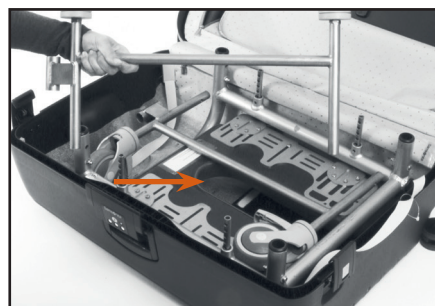

Place the legs in the suitcase.

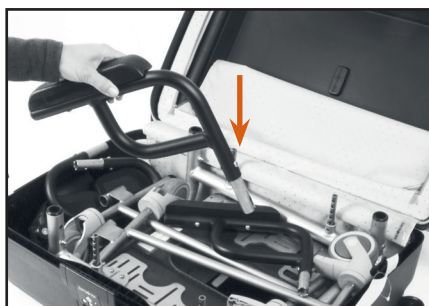

Place the armrests in the suitcase in between the legs.

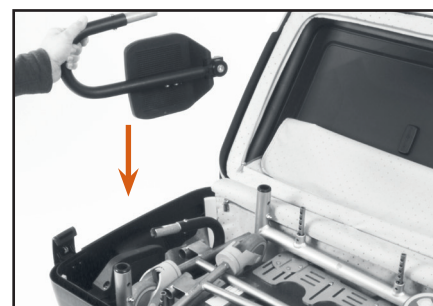

Place the legrests behind the frame in the suitcase.



Place any other objects in the suitcase (f.e. buckets, bedpans).



When applicable, place the 20" rear wheels in the cover of the suitcase. The wheels should overlap and be placed on one side of the cover.

✓ Remove soap and dirt regularly by using a soft soap. Do not use any caustics like chlorine or alcohol. It is possible to clean the frame with Antikal.

✓ Neoprene and Platilon are comfortable but sensitive materials. Be careful with sharp objects like zippers, buttons, lifts, etc.