

BACKREST ADJUSTMENT



Loosen the knobs both left and right from the backrest.




Adjust the backrest to a desired position and tighten the knobs again.

ARMRESTS




Slightly lift the armrests to swivel them around. When placing them back, make sure the armrests will be tightened by pushing them down well enough.

SEAT POSITION

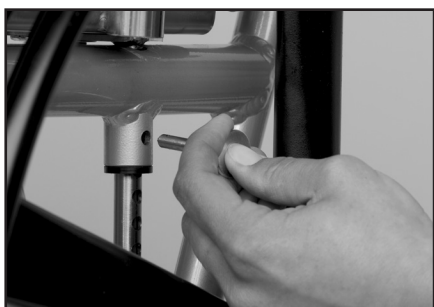

Remove the 4 quick-release pins from under the seat.

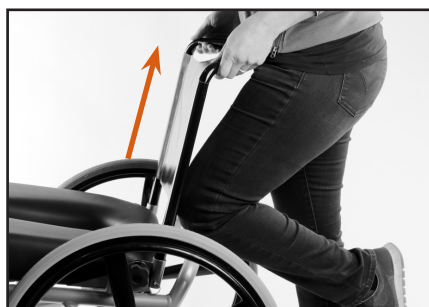

Lift the backrest and seat to a desired position. Replace the quick-release pins.

ANGULAR ADJUSTMENT



Push the seat max. one pin up on the front of the seat comparing to the back, to create an angular adjustment. Replace the pins at front and push the back down.

FOOTREST ADJUSTMENT



Unscrew the bolts with an Allan key nr. 4 and a spanner nr. 10.



Place the footrest in the desired position and screw it tight again.

ANGLE FLIP UP FOOTREST



The flip up footrests are having the possibility to change the angle. Use an Allan key nr. 5.

SWING AWAY LEGRESTS



The footrests of the swing away legrests (life special and junior special) can be adjust by unscrew the screws by using an Allan key nr. 4.



For more adjustment of the leg- and footrests, the legrestreceiver on the frame can be adjusted with an Allan key nr. 4 and spanner nr. 10.

BRAKES



The brakes can be locked or unlocked by pushing or pulling. The brakes can be tightened when needed by using an Allan key nr. 5.



Adjust the seat in the lowest position by removing the quick release pins. Push the seat down and replace the pins.



Remove the armrests by lifting them out of their position.



In case of a life special or junior special remove the legrest by lifting them out of their position.



Loosen the knobs of the backrest (left and right) and lay the backrest down.



Remove the backrest from the receivers. The backrest has to be turned. The bend is directed downwards. Replace the backrest and tighten the knobs.



Press the quick-release axle of the wheel to remove the wheel. Remove the axles from the wheel before transport, to insure they don't bend.



MAINTENANCE

- ✓ Remove soap and dirt regularly by using a soft soap. Do not use any caustics like chlorine or alcohol. It is possible to clean the frame with Antikal.
- ✓ The seat can be removed to clean the seat and the high-low system properly.
- ✓ Spray the quick release axles of the wheels with silicone spray once in every three months.

- ✓ Spray the quick-release pins of the seat and backrest with silicone spray once in every three months.
- ✓ Neoprene and Platilon are comfortable but sensitive materials. Be careful with sharp objects like zippers, lifts, etc.
- ✓ Replace the seat over time. Normally, but depending on the frequency and intensity of the use, it can be used for 3 years. After 3 years, Neoprene and Platilon become porous and water soaks into the underlying polyether.



IMPORTANT WITH TRANSPORT

- ✓ Keep the quick release axles pressed while removing or replacing the wheels. If not: the bearings can be pushed out and the wheel can't be replaced.
- ✓ If the wheel is not positioned properly, the bearing is probably slightly pushed out (the deviation is often slight, but can have a big impact). Tap with a hammer (or other heavy object) on the bearing in the wheel. The bearing will be repositioned and the wheel can be replaced properly.