

## DUKE FRAME – DEPTH VARIANT

### BACKREST POSITION

The wheelchair is, by default, assembled with the backrest fixed to **position 1** (the more forward of the two).

On specific request to write on the order form as note, but also as possible modification after sale, the backrest can be fixed to **position 2** that is 1.5 cm behind position 1.




### SETTINGS

Backrest fixed to <b>position 1</b> . (default assembly)	105 mm	ACTIVE	120 mm	Backrest fixed to <b>position 2</b> . (assembly on request or after sale)
	90 mm	STANDARD	105 mm	
	75 mm	PRUDENTIAL	90 mm	

### SEAT DEPTH (PS) – FRAME LENGTH

Frame length according to PS requested on the order form.

PS cm.	Frame length	L cm
35 - 37.5	SHORT	55
40 - 42.5	MEDIUM	60
42.5 - 45 - 50	LONG	65

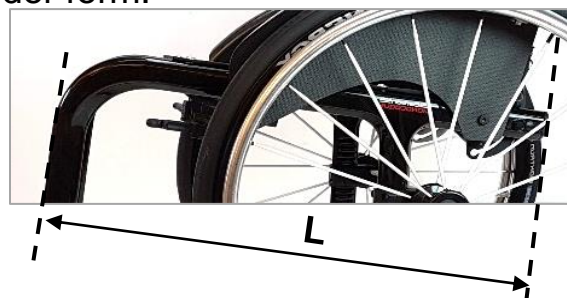

### CHART OF POSSIBILITIES

PS cm.	Frame length	Backrest fixed to <b>position 1</b>		Backrest fixed to <b>position 2</b>	
		Real PS	Settings	Real PS	Settings
35	SHORT	36.5	75 / 90 / 105	39.0	90 / 105 / 120
37.5		39	75 / 90 / 105	40.5	90 / 105 / 120
40	MEDIUM	41.5	75 / 90 / 105	43.0	90 / 105 / 120
42.5		44.0	75 / 90 / 105	45.5	90 / 105 / 120
45	LONG	46.5	75 / 90 / 105	48.0	90 / 105 / 120
47.5		49.0	75 / 90 / 105	50.5	90 / 105 / 120
50		---	---	50.5	90 / 105 / 120

Note: **real PS** = from front side of seat canvas to front side of backrest tube.