

A photograph of four teenagers outdoors. In the foreground, a young woman with blonde hair in a ponytail, wearing a light blue denim shirt and jeans, sits in a bright blue QUICKIE wheelchair. She is smiling and looking back over her shoulder. Behind her, another young woman with long brown hair sits on a red mobility scooter, also smiling. To the right, a young man in a black t-shirt and jeans stands, holding a long wooden stick. In the background, another person is crouching on the grass. The setting is outdoors with a light blue building and green foliage in the background.

# LIFE FT & LIFE RT

FOR TEENS ON THE MOVE



COOL, INDIVIDUAL,  
MODERN.

# TWO WHEELCHAIRS – ONE FAMILY

Live life to the full! Meeting your friends or going to school? Whatever your plans for the day, the Quickie Life FT and Life RT are two modern and fresh wheelchairs that combine a robust frame with cool features to suit and active lifestyle.

## So why does the Quickie Life Teens work?

- Cruise with ease – A choice of 22" or 24" wheels to get the roll you want.
- Maximise your efficiency – Adjustable centre of gravity set-up means that the chair can be fine tuned for ultimate performance or as your skills improve.
- Easy to reach hand rims – Adjustable camber option from 0° to 6° with no additional parts needed makes it possible to get the handrims in the most easy to use and ergonomic position for any user.
- Confidence in your stability – Choose an abducted frame without rigged castors to give more forward stability and easier forward transfers.



## LIFE FT FOLDING TEENS

Fixed or swing-away front frame



## LIFE RT RIGID TEENS

Fixed or swing-away front frame



→ **Abduction frame for Life FT & Life RT**

This frame version offers 2.5cm abduction on each side, giving increased forward stability - great for smaller teens.

# LIFE FT & LIFE RT – EXPRESS YOURSELF

---

## **Personalise to your style and performance**

- Individual to you – Choose from more than 30 different colours, three different accessories colour packs and spoke guards to get a chair that fits you and your personality.
- Personalise your options to your LIFE! – Choose from a host of side guards, armrests, leg rests, footplates and upholstery to get a chair that fits your lifestyle and needs.
- Wicked adjustments for maximum performance – Each chair is packed with adjustments so that you can fine-tune as your skillz improve or if you grow.



→ **Wide camber range for Life FT & Life RT**

Camber can improve lateral stability and position the wheels better for smaller users - 0° to 6° adjustment.





→ **The innovative Carbotecture fork**

Made out of stiff high-tech material and extremely lightweight, also available as an anodised pack in three different colours.



→ **Carbon platform footplate**

Choose a lightweight carbon footplate and sideguard to express your individuality.

# LEONARDO'S LIFE



→ LIFE RT - the sporty rigid wheelchair



→ LIFE FT - the robust folding wheelchair

## Options and features of the Quickie Life FT and Life RT – get the Life you want.



### // ABDUCTION FRAME

This frame version offers 2.5cm abduction on each side. Great for forward stability and abducted legs.



### // ANGLE ADJUSTABLE BACK

This angle adjustable backrest provides an adjustment from a closed to open angle in 4° increments.



### // SEAT HEIGHT ADJUSTABLE IN SEVEN POSITIONS

Camber from 0° to 6° and four centre of gravity positions (from the very active 8.3cm to the more stable 2.3cm). Get the correct wheel position for your Life!



### // GROW IN LIFE

Depth adjustable seat upholstery offers growth in the seat canvas (for the Quickie Life RT).



### // THE UNIQUE SAFARI WHEEL LOCK

With its intuitive handling, the safari sideguard wheel lock helps you stop quickly and safely.



### // IT'S ALL ABOUT ERGONOMICS

The push bar can be easily adjusted in angle and height for an enhanced ergonomic position for parents and carers.



### // CARBON PLATFORM FOOTPLATE

Lightweight and stylish footplate with angle and depth adjustment.



### // YOUR LIFE

Choose from more than 30 different colour options free of charge.

# TECHNICAL DATA

	LIFE FT	LIFE RT
Max. user weight:	100 kg	100 kg
Seat width:	28 – 50 cm	28 – 48 cm
Seat depth:	32 – 46 cm	32 – 46 cm
Seat height front:	40 – 53 cm	36 – 50 cm
Seat height rear:	40 – 50 cm	36 – 45 cm
Backrest height:	25 – 47.5 cm	25 – 45 cm
Backrest angle:	78° to 102° in 4° increments	78° to 98° in 4° increments
Centre of gravity (COG):	2.3 – 8.3 cm	2.3 – 8.3 cm
Camber:	0° – 6°	0° – 6°
Fixed frame front:	75°; 75° with abduction; 85°; 85° with inset	75°; 75° with abduction; 85°; 85° with inset
Swing away frame front:	80° and 70°	80° and 70°
Chair weight:	from 10.5 kg	from 10 kg
Frame colours:	32	32
Crash tested:	yes	yes

Find out more today!

Visit **[www.SunriseMedical.co.uk](http://www.SunriseMedical.co.uk)** for further information on product options, or contact your local dealer or representative.

## Get SOCIAL with QUICKIE and JAY

Keep up-to-date with the latest news on Quickie and JAY products through our Facebook page and YouTube channel:



[www.facebook.com/QuickieWheelchairs](https://www.facebook.com/QuickieWheelchairs)



[www.youtube.com/quickiewheelchairseu](https://www.youtube.com/quickiewheelchairseu)



PB\_QUICKIE\_LIFE\_FT\_&\_LIFE\_RT\_REV2

## Sunrise Medical Limited

Thorns Road, Brierley Hill, West Midlands, DY5 2LD

Tel: +44 (0) 845 605 66 88 Fax: +44 (0) 845 605 66 89

**[www.SunriseMedical.co.uk](http://www.SunriseMedical.co.uk)**