



zipie®
by QUICKIE™



Salsa

Explore

your world



The Zippie Salsa and Salsa M deliver great outdoor performance and excellent indoor manoeuvrability along with multiple seating and powered options. This ensures your child has the perfect combination of function, performance and fun!

Choose the rear wheel drive Zippie Salsa for great all-round performance.

Zippie® Salsa



Great Outdoor Performance!

Thanks to the Zippie Salsa's powerful motors, 60Ah batteries and chunky wheels, performance outdoors is outstanding.

Capable of climbing over obstacles of up to 10cm (4") and tackling inclines of up to 8° (14%), the Zippie Salsa can travel up to 32km (20m) in comfort.

Excellent Indoor Manoeuvrability!

The Zippie Salsa's ultra-compact base and super-low seat height make access into vehicles, around confined spaces and under tables easy.





When the ultimate in manoeuvrability is required, select the mid wheel drive Zippie Salsa M.

Zippie® Salsa M



Outstanding Indoor Manoeuvrability!

The Zippie Salsa M's mid wheel drive base delivers the best possible indoor performance thanks to its ultra-compact turning radius and small footprint. A narrow width of just 60cm (24") and low seat height make access very easy.

Great Outdoor Performance!

The Zippie Salsa M performs as well outdoors as it does indoors, featuring a unique suspension system that delivers a smooth, safe and stable ride even when travelling over slopes and cambers. It's also capable of overcoming obstacles of up to 7.5cm (3") and it can travel up to 26km (16m) on a full charge.

Shown with optional comfort seating system

A Range of Powered Options



Powered options available on the Salsa and Salsa M:

- Powered seat tilt (0° to 30° or -5° to 25°)
- Powered seat lift and tilt (30cm of lift, 0° to 30° of tilt)
- Powered seat recline (0° to 30°)

Thanks to the modular nature of the Salsa and Salsa M, powered options can also be added after purchase via retro-fit kits. That means if your child's needs change, the Salsa can adapt easily and cost effectively to them.

Electronics

- PG Drives Technology VR-2 70Ah controller as standard
 - Controls up to two powered functions
 - Dual control option
 - Compact joystick
- Optional PG Drives Technology R-net controller offers the following additional functionality:
 - Controls up to six powered functions
 - LCD colour joystick option
 - Exclusive 'assignable buttons' short-cut feature
 - Dual control option
 - Special controls interface module (OMNI) for
 - Environmental controls
 - PC mouse control via R-net Bluetooth module






Seating that grows with your child

The standard Salsa/Salsa M seating system offers built-in growth potential for seat depth, width and height. It also features height and depth adjustment in the armrests and angle and depth adjustment in the footplates. The seat and backrest also offer angle adjustment for improved comfort.



Swing-away and removable tray table

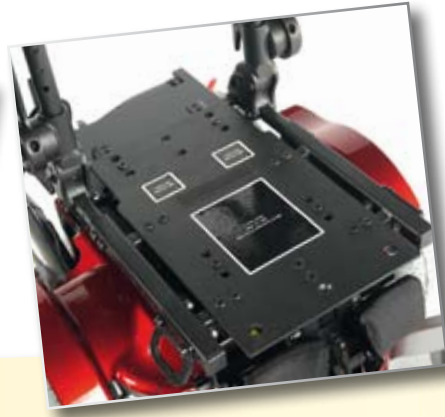

Predefined areas for modified seat fixations



Optional swing-away laterals with comfort seat

- Fully adjustable standard seating system that grows with your child: 12 x 12" to 16 x 16"
- Optional comfort seat cushion and back for improved postural support and comfort
- Seat frame is easily adaptable for Jay® or other third party seating systems
- Wide range of seating options and accessories for perfectly tailored function and comfort

90° swing-away removable hangers



Center mount leg rest with angle adjustable and flip-up footboard (optional)



Standard flip-up armrests and foot rests for easy accessibility



Technical Specifications




Seating system adjustments

Seat depth:	Adjustable from 30 to 40cm (12 to 16") (in 2cm increments)
Seat width:	Adjustable from 30 to 40cm (12 to 16") (in 2cm increments)
Seat height*:	37 - 46cm (14.5 - 18") [Salsa R] 40 - 53cm (16 - 21") [Salsa M]
Seat angle:	Adj. 0°, 3°, 6°, 9°
Backrest height:	Adj. 38.5cm (15"), 41cm (16") and 43.5cm (17")
Backrest angle:	Adj. -4°, 0°, 4°, 8°, 12°, 16°
Armrest height:	Adj. from 17 to 27.5cm (6.5 to 11")

Salsa

Chair width

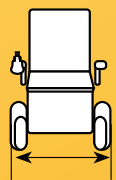

58cm (23") (12" wheels)
60cm (24") (14" wheels)

Chair length
std. legrests



114cm
(45")

Max safe slope



8° (14%)

Max user weight



75kg** / 12st


Max kerb climb



10cm (4")
(with kerb climber option)

Salsa M

Chair width

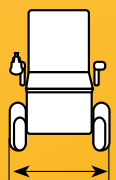

60cm
(24")

Chair length
std. legrests



99cm
(39")

Max safe slope



8° (14%)

Max user weight



75kg** / 12st

Max kerb climb



7.5cm (3")

Technical data

	Salsa R	Salsa M
Speed:	6 / 10kph (4 / 6mph)	6 / 10kph (4 / 6mph)
Maximum range:	32km (20miles) (ISO 7176-4)	26km (16miles) (ISO 7176-4)
Batteries:	2 x 60Ah	2 x 50Ah
Turning radius:	85cm (33")	60cm (24")
Controllers:	VR2 (R-net option)	VR2 (R-net option)
Overall weight:**	113 - 165kg	113 - 165kg



Sunrise Medical Limited

Thorns Road
Brierley Hill
West Midlands
DY5 2LD
Tel: +44 (0)845 605 66 88
Fax: +44 (0)845 605 66 89
www.SunriseMedical.co.uk
enquiries@sunmed.co.uk

*Seat height depends on the chosen options, please refer to order form for full details

**Please note: Total weight of the powerchair depends on the configuration

*** Plus additional 15kg for third party seating system